

Chemical resistance guide for Finesse & Finesse PF

It is important to note that the performance of these chemicals may be different if mixtures of chemicals are involved.

Chemical	EN374 performance level
Acetic acid 10%	3
Citric acid	6
n-Heptane	0
Hydrochloric acid 32%	3
Isopropyl alcohol	0
Methanol	0
Nitric acid 40%	2
Phosphoric acid 85%	6
Propan-1-ol	0
Sodium hydroxide 10%	6
Sodium hydroxide 50%	6
Sodium hypochlorite 14%	6
Sulphuric acid 30%	6
Sulphuric acid 96%	1

EN374 Performance Levels

Performance Level	1	2	3	4	5	6
Breakthrough time (mins)	>10	>30	>60	>120	>240	>480

The information contained here is advisory and is intended to guide and inform only. This information is based on the view of BM Polyco Ltd in light of all available data and is not subject to any guarantee on our part regarding the performance of a particular glove style. As it is impossible to test every situation encountered and all chemical and mixtures of chemicals, the user is advised to test the particular glove style before and periodically during use.

